

Anti-Slip Platform



The Anti-Slip platform is used to secure the BxBoard during tissue loading. It prevents the BxBoard from sliding or moving, protecting both the user and the patient tissue.

Procedure

1. Place the Anti-Slip Platform on the workstation of the procedure room. This could be a counter, cart, stand, or whatever would typically be used to hold the tissue transport containers or formalin jars.
2. Orient the Platform as in the picture shown above.
3. Place the BxBoard onto the Anti-Slip Platform with the BxBoard lanes facing toward the user.
4. Load the BxBoard with needle core biopsies according to procedure.
5. Close and seal the BxBoard according to procedure.
6. Clean the Anti-Slip Platform.

Cleaning Instructions

1. Clean the Anti-Slip Platform after each procedure.
2. The Platform can be cleaned using dish soap and water.
3. Use a cloth or paper towel to gently wipe the Platform dry.
4. The Platform has no special storage instructions.